



Eye Drops



Moisturising

For dry, irritated, tired eyes

Suitable for contact lens users

Moisturising Eye Drops

For relief of dry, irritated or tired eyes. Provides quick moisturisation of the eye surface.

Helps soothe and maintain lubrication of the eye.

Suitable for contact lens users and those with sensitive eyes.

- 10 ml sterile solution
- No preservatives
- Use within 6 months of opening

Revitalising and moisturising

Eyes can sometimes feel dry, irritated or tired because of computer use, heating or air-conditioning, age or

through wearing contact lenses. A.Vogel Moisturising Eye Drops contain ingredients to refresh the eyes, helping to keep them moist whilst maintaining a liquid film.

Unique dropper bottle

A.Vogel Moisturising Eye Drops come in a unique dropper bottle. The easy-to-use dropper system releases just one drop at a time for ease of application.


Dosage and use

Adults: 1 drop in each eye 4-5 times daily. The drops can be used for a maximum of 30 consecutive days. If your symptoms persist for more than 3 consecutive days without any improvement, or get worse, or if eyes become inflamed, you must contact your optician or healthcare professional.


Children from the age of 12 years: 1 drop in each eye 1-2 times daily.

A.Vogel Moisturising Eye Drops can be used by children over 12 if it has been established by an optician or healthcare professional that their eye problems do not require a different, medical treatment. The maximum duration of use in children must be determined by an optician or healthcare professional. A.Vogel Moisturising Eye Drops must be administered by an adult. Children may experience the application of eye drops to be unpleasant. Therefore, be alert to unexpected movements of the child during the application of the drops, in order to prevent the tip of the bottle touching the eye.

The dropper bottle contains 10 ml sterile solution (about 300 drops). Please note, there will always be a small amount of liquid left in the bottle. We recommend that this product is not shared with other people.

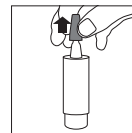


1 Remove the coloured cap from the bottle.

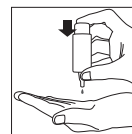

2 First time use: Hold the bottle upside down over your hand and squeeze the base several times until the first drop is released. The dropper bottle is now ready for use.



3 Tilt the head back slightly and pull the lower eyelid downwards. Place the dropper above the eye. Holding the bottle with your thumb and middle finger, press the base of the bottle with your index finger.

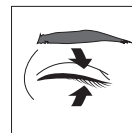

4 Close your eye so that the fluid spreads evenly over the eye's surface. Replace the cap on the bottle, pressing firmly.

What you need to know before you use A.Vogel Moisturising Eye Drops

Do not exceed the stated dose.



Use during pregnancy or breastfeeding

A.Vogel Moisturising Eye Drops can be used as directed at the dose stated.

Do not use if you are

Hypersensitive to any of the ingredients.

Tolerability

The unique bottle design keeps the contents sterile without using preservatives, ensuring the product once open, can be used frequently.

Ingredients

Hyaluronic acid (1 mg/ml), *Euphrasia officinalis* (Eyebright), sodium chloride, boric acid, disodium tetraborate decahydrate, water.
No preservatives. Suitable for vegans.

Storage

Keep out of the sight and reach of children. Store between 8°C and 25°C.
Use within 6 months of opening. Do not use after the expiry date.

Packaging

A.Vogel Eye Drops are available as:

A.Vogel Moisturising Eye Drops	For dry, irritated and tired eyes	10 ml
A.Vogel Extra Moisturising Eye Drops	For very dry and irritated eyes	10 ml

Personal advice

For further details or healthcare advice go to www.avogel.co.uk, or call the A.Vogel Customer Helpline on 0845 608 5858 or send an email to enquiries@avogel.co.uk.

Tips

- Use natural light as much as possible. Do not work at a computer screen for long periods at a time. You can avoid this by taking regular breaks.
- Do not wear contact lenses for longer than the manufacturer's recommendation.
- Consider wearing sunglasses to avoid irritation from sunlight.
- Avoid air-conditioning where possible. Choose to open a window instead.
- Don't watch too much TV.

Care for the environment

A.Vogel strives to minimise the impact on the environment. As far as possible our packaging materials are produced using environmentally friendly methods. These can be incinerated without any harmful impact on the environment or they are suitable for recycling.

The text of this leaflet was most recently revised on 15 May 2017.



Biohorma B.V.
J.P. Broekhovenstraat 16
8081 HC Elburg The Netherlands

Distributed in the UK and Ireland by:

Bioforce (UK) Ltd
2 Brewster Place
Irvine
KA11 5DD
UK

CE 0481



STERILE A

