

# A.Vogel

## Pollinosan Eye Drops

### HAYFEVER



### For red and itchy eyes

Suitable for  
contact lens users

#### Hayfever Eye Drops

For quick relief of red and itchy eyes due to hayfever. A.Vogel Pollinosan Hayfever Eye Drops offer protection against pollen which may trigger a hayfever response. Suitable for contact lens users.

- 10 ml sterile solution
- No preservatives
- Use within 6 months of opening

#### With hyaluronic acid

A.Vogel Pollinosan Hayfever Eye drops contain hyaluronic acid which helps soothe red and itchy eyes.

The hyaluronic acid in A.Vogel Pollinosan Hayfever Eye Drops is produced from a vegetable source. It does not contain genetically modified soy and is not of animal origin.

#### Unique dropper bottle

A.Vogel Pollinosan Hayfever Eye Drops come in a unique dropper bottle. The easy-to-use dropper system releases just one drop at a time for ease of application. The unique bottle design keeps the contents sterile without using preservatives, ensuring the product can be used frequently, once open.

The eye drops can be used within 6 months of opening.

Contact the A.Vogel Helpline if you require any further information.  
UK: 0845 608 5858 or send an email to [enquiries@avogel.co.uk](mailto:enquiries@avogel.co.uk)  
IE: 1 890 930 070 or send an email to [enquiries@avogel.ie](mailto:enquiries@avogel.ie)

#### Use with contact lenses

If you wear contact lenses, you do not need to take them out before using these eye drops.  
Suitable for contact lens users.

#### Dosage and use

**Adults:** 1-2 drops, as needed, in each eye up to 5 times daily. The drops can be used for a maximum of 30 consecutive days. If your symptoms persist for more than 3 consecutive days without any improvement, or get worse, or if eyes become inflamed, you must contact your optician or healthcare professional.


**Children over 7 years:** 1 drop in each eye 1-2 times daily, after consulting an optician or healthcare professional.

A.Vogel Pollinosan Hayfever Eye Drops can be used by children aged 7 and over if it has been established by an optician or healthcare professional that their eye problems do not require a different medical treatment. The maximum duration of use in children must be determined by an optician or healthcare professional, but it may not exceed 30 days. A.Vogel Pollinosan Hayfever Eye Drops must be administered by an adult. Children may experience the application of eye drops to be unpleasant. Therefore, be alert to unexpected movements of the child during the application of the drops, in order to prevent the tip of the bottle touching the eye.


If using additional eye treatments with other eye drops, leave an interval of at least 30 minutes before use. Ointments should be applied in the conjunctival sac (inside the lower eyelid) at least 30 minutes after applying A.Vogel Pollinosan Hayfever Eye Drops.

The dropper bottle contains 10 ml sterile solution (about 300 drops).

Please note, due to the design of the dropper bottle, there will always be a small amount of liquid left in the bottle. We recommend that this product is not shared with other people.

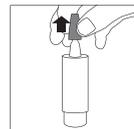


1 Remove the coloured cap from the bottle.

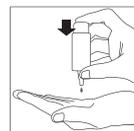

2 First time use: Hold the bottle upside down over your hand and squeeze the base several times until the first drop is released. The dropper bottle is now ready for use.



3 Tilt the head back slightly and pull the lower eyelid downwards. Place the dropper above the eye. Holding the bottle with your thumb and middle finger, press the base of the bottle with your index finger. The eye drops should be applied in the conjunctival sac (inside the lower eyelid).

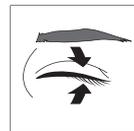

4 Close your eye slowly so that the fluid spreads evenly over the eye's surface.  
Replace the cap on the bottle, pressing firmly. Close the bottle tightly with the cap after use.

#### What you need to know before you use A.Vogel Pollinosan Hayfever Eye Drops

Do not exceed the stated dose.

### Use during pregnancy or breastfeeding

There is no information available on the use during pregnancy or breastfeeding. Consult your doctor or healthcare professional before using A.Vogel Pollinosan Hayfever Eye Drops.

### Do not use in case of

Hypersensitivity to any of the ingredients.

### Possible side-effects

Occasionally, hypersensitivity reactions (burning and local pain) may occur in the eye during administration.

### Ingredients

Hyaluronic acid (1 mg/ml), *Chamomilla recutita* (chamomile), Sodium chloride, Boric acid, Disodium tetraborate decahydrate, Sterile water.

May contain traces of soy.

No preservatives. Suitable for vegans.

### Storage

Store between 8°C and 25°C. Keep out of the sight and reach of children.

Use within 6 months of opening. Do not use after the expiry date.

### Tips

- Keep doors and windows closed on hot and windy days to prevent pollen particles entering your house.
- Air rooms early in the morning, when it is still damp outside.
- Consider wearing sunglasses to protect the eyes from pollen.
- Go for a walk when the pollen count is low.

### Care for the environment

A.Vogel strives to minimise the impact on the environment. As far as possible our packaging materials are produced using environmentally friendly methods. These can be incinerated without any harmful impact on the environment or they are suitable for recycling.

 0482

 8°C - 25°C

 STERILE A



 HÄLSA Pharma GmbH  
Maria-Goeppert-Strasse 5  
D-23562 Lübeck, Germany

### Distributor:

**UK:** A.Vogel Ltd, 2 Brewster Place, Irvine, Ayrshire  
KA11 5DD

**IE:** A.Vogel Ireland Ltd, Unit 3D, Killeen Road, Dublin 10  
D10 TY20

The text of this leaflet was most recently revised in July 2020.

### Product range

A.Vogel Eye Drops are available as:

A.Vogel Moisturising Eye Drops*	For dry, irritated and tired eyes	10 ml
A.Vogel Extra Moisturising Eye Drops*	For very dry and irritated eyes	10 ml
A.Vogel Pollinosan Hayfever Eye Drops*	For red and itchy eyes	10ml

\* medical device - read instructions before use.



Manufacturer



Best before end



Batch number



Read instructions before use



Store between 8°C and 25°C



Sterile device



Recycle



Read directions for use regarding warnings and precautions



CE symbol with notified body number